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These instructions describe the lamp replacement for the KARL STORZ light sources XENON 175, XENON 300 and D-LIGHT/AF/C.
Note: The lamp modules are available in two models: Version 1 (V1) and Version 2 (V2).
Note XENON NOVA® 175/300: Replacing the lamp is described in the XENON NOVA® 175/300 instruction manual.

Tools required

- Slot head screwdriver
- Torx screwdriver, size 25
- Hex key 7/64"
- Thermo-lubricant

Warning: Danger of explosion. The lamp is filled with XENON gas at very high pressure. It must therefore be handled with appropriate care and must only be replaced by skilled electrical engineers. Strong mechanical forces could cause the lamp to break.
Wear safety goggles! We expressly recommend wearing protective clothing (face protection, gloves, etc).

How to proceed

V2: Undo the Torx screws of the heat sink clips.

V1: Release the clip with the aid of the slot head screwdriver.

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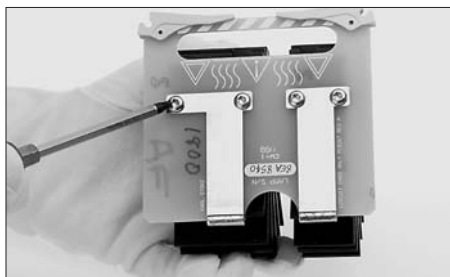
Replacing the XENON lamp in the lamp module

Completely undo and remove the two Torx screws of the wide* heat sink (without light outlet) with the size 25 Torx screwdriver. The narrow* heat sink (with light outlet opening) remains screwed on the PCB as a guide.

*** Note:** In the lamp module of the XENON 175 the heat sinks have the same width, therefore the differentiation by "light outlet". Next carefully pull the wide* heat sink (with the lamp) away from the PCB.

Note: If the heat sink cannot be removed, proceed as follows.

Place a slot head screwdriver in the gap of the narrow* heat sink (with the light outlet) and turn it slightly in order to loosen the thermo-lubricant where it has adhered to the heat sink.

Undo and remove the three hex screws (7/64") on the rear of the wide* heat sink (without light outlet), paying attention to the spring lock washers. Pull out the lamp.

Note: Follow the respective national ordinances when disposing of the defective lamp. The defective lamp can be sent back to KARL STORZ for disposal. It must be dispatched in the original packaging.

Caution: Danger of explosion! Avoid turning the heat sink. The pin of the Xenon lamp (see arrow) must not be damaged.

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Thoroughly clean both heat sinks with cotton swabs and a small amount of spirit or similar.

Apply the thermo-lubricant onto the inside of the lamp receptacle of the heat sinks and onto the metal surfaces of the new lamp.

Only apply the thermo-lubricant in a thin film (a metallic contact should still be possible).

Place the lamp with protective cap on the table/pad. Correctly position the wide* heat sink (without light outlet) over the lamp (when doing so ensure that the hex screws (7/64") are aligned properly and that the pin is positioned correctly!). Allow the heat sink to slide over the lamp (do not use force, the weight of the heat sink is sufficient).

Next position the screw holes of the heat sink and lamp exactly over each other.

Do not touch the glass surface of the lamp. Clean with alcohol if necessary.

Place the spring lock washers on the three hex screws (7/64") and then screw the wide* heat sink (without light outlet) and lamp together. Now coat the front metal surface of the lamp with thermo-lubricant (see arrow).

Do not touch the glass surface of the lamp. Clean with alcohol if necessary.

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
Position the PCB with the narrow* heat sink (with light outlet) on the table so that it rests on the table with the PCB pointing vertically upward. Allow the wide* heat sink with mounted lamp to slide in (do not use force). For this, the contact surface of the heat sink must be aligned with the PCB.

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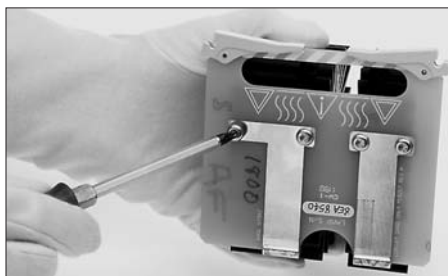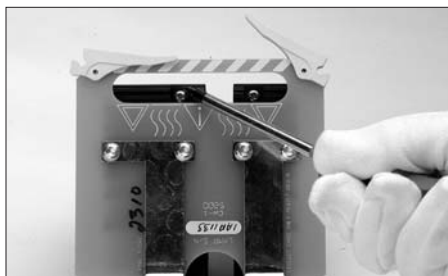
Screw the wide* heat sink (without light outlet) onto the PCB again.

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
V2: Finally, mount and fasten from below the two screws which press the heat sinks together.

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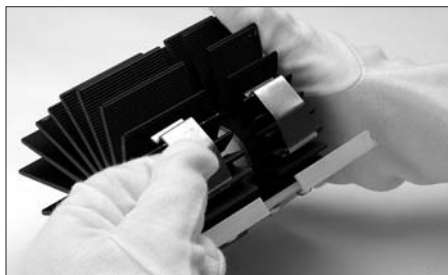V1: Finally, mount the two clips on the heat sinks.

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